

# Bard Orbiter™

16- & 24-pole woven diagnostic catheter




## The woven advantage for simplified atrial mapping

Orbiter woven atrial mapping catheters offer exceptional stability, easy positioning and superb signal quality. Bard's unique woven material allows the Orbiter catheter to soften at body temperature to conform to the cardiac wall. The result is excellent wall contact and crisp, clear electrograms.

The Orbiter catheter provides the comprehensive coverage needed to localize the origin of most atrial tachycardias.


Exceptional stability against the atrial wall enables the Orbiter catheter to deliver consistently crisp electrograms.

A choice of curves accommodates variations in your patients' atrial anatomy.

-  Woven, multi-pole construction assures reliable performance in a wide variety of clinical settings.
-  Woven bioflexibility assists in positioning the curve for recording along the septum, roof and lateral wall of the right atrium.
-  Bard offers the Orbiter catheter in standard and higher-torque models to fit a wide range of user preferences.



The Orbiter catheter curves cover the most common ectopic foci, allowing rapid identification of the site of earliest activation.

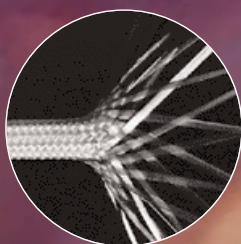

To produce woven bioflexibility, the woven core is impregnated with a proprietary polymer in an environment of strictly-controlled temperature.



Orbiter-Small, Orbiter-Large and Orbiter Milwaukee curves provide outstanding diagnostic flexibility.

# Ordering Information

## Bard Standard 16-Pole Orbiter™ Woven Catheters

Item No.	French	No. of Electrodes	Curve	Electrode Spacing	Tip Electrodes	Proximal Electrodes	Recording Span	Usable Length
161601	6F	16	Small 60	1.5,1mm	2mm	1mm	6.0cm	110cm
161602	6F	16	Small 60	1.5,1mm	2mm	1mm	6.0cm	80cm
161603	6F	16	Damato	1.5,1mm	2mm	1mm	6.0cm	80cm

## Bard Standard 24-Pole Orbiter Woven Catheters

Item No.	French	No. of Electrodes	Curve	Electrode Spacing	Tip Electrodes	Proximal Electrodes	Recording Span	Usable Length
242401	6F	24	Milwaukee	2.4,2mm	2mm	1mm	9.3cm	110cm
242402	6F	24	Orbiter-Small	2.4,2mm	2mm	1mm	9.3cm	110cm
242403	6F	24	Orbiter-Small	2.7,2mm	2mm	1mm	12.6cm	110cm
242404	6F	24	Orbiter-Large	2.7,2mm	2mm	1mm	12.6cm	110cm
242405	6F	24	Orbiter-Small	2.9,2mm	2mm	2mm	17.1cm	110cm
242406	6F	24	Orbiter-Large	2.9,2mm	2mm	2mm	17.1cm	110cm

## Bard 24-Pole High-Torque Orbiter Woven Catheters

Item No.	French	No. of Electrodes	Curve	Electrode Spacing	Tip Electrodes	Proximal Electrodes	Usable Length
242411	6F	24	Orbiter-Small	2.4,2mm	2mm	1mm	110cm
242412	6F	24	Orbiter-Small	2.7,2mm	2mm	1mm	110cm
242413	6F	24	Orbiter-Large	2.7,2mm	2mm	1mm	110cm
242414	6F	24	Orbiter-Small	2.9,2mm	2mm	1mm	110cm
242415	6F	24	Orbiter-Large	2.9,2mm	2mm	1mm	110cm
242416	6F	24	Milwaukee	2.4,2mm	2mm	1mm	110cm

## Bard Orbiter Cable Information

Item No.	Use	Length	Description
200774P	Single Use	125cm	Protected 8-Pin Easy-Mate™

**Bard Electrophysiology Division**  
USA  
Tel: 978-441-6202  
800-824-8724 (toll free w/in the US)  
Fax: 978-323-2222  
877-281-1492 (toll free w/in the US)

**Bard Asia Pacific Division**  
USA  
Tel: 908-277-8000  
Fax: 908-277-8090

**Bard Australia, Pty Ltd**  
AUSTRALIA  
Tel: 61-2-8875-4000  
Fax: 61-2-8875-4050

**Bard Benelux, N.V.**  
BELGIUM  
Tel: 32-0-1428-6950  
Fax: 32-0-1428-6955

**Bard Canada, Inc**  
CANADA  
Tel: 905-275-8000  
Fax: 905-275-3383

**Bard International, Inc**  
CHINA  
Tel: 86-10-8391-3282  
Fax: 86-10-8391-3285

**Laboratories Bard SA**  
FRANCE  
Tel: 33-1-39-30-5858  
Fax: 33-1-39-30-5822

**C. R. Bard Germany, GmbH**  
GERMANY  
Tel: 49-721-9445-265  
Fax: 49-721-9445-111

**Bard International**  
INDIA  
Tel: 91-22-612-3783  
Fax: 91-22-612-3821

**Bard S.p.A.**  
ITALY  
Tel: 39-06-52-493-1  
Fax: 39-06-52-958-52

**Medicon, Inc**  
JAPAN  
Tel: 6-6203-6546  
Fax: 6-6203-1516

**Bard International, Inc.**  
KOREA  
Tel: 82-2-719-3080  
Fax: 82-2-719-3081

**Bard Latin America**  
USA  
Tel: 908-277-8000  
Fax: 908-277-8090

**Bard Medical Systems Norden**  
SWEDEN  
Tel: 46-42-386-000  
Fax: 46-42-386-010

**C. R. Bard Portugal, LDA**  
PORTUGAL  
Tel: 351 21 319 03 30  
Fax: 351 21 319 03 39

**Bard Singapore Pte. Ltd.**  
SINGAPORE  
Tel: 65-250-7311  
Fax: 65-353-7311

**Bard De Espana, SA**  
SPAIN  
Tel: 34-93-253-7800  
Fax: 34-93-253-7834

**Bard Medica, SA**  
SWITZERLAND  
Tel: 41-22-885-9618  
Fax: 41-22-885-9635

**Pacific Health Care Co., Ltd**  
TAIWAN  
Tel: 886-2274-07-891  
Fax: 886-2277-75-576

**Bard Limited**  
UNITED KINGDOM  
Tel: 44-1-293-527-888  
Fax: 44-1-293-552-428



Bard Electrophysiology  
55 Technology Drive • Lowell, MA 01851 USA  
Tel: (978) 441-6202 • Fax: (978) 323-2222  
[www.BARDEP.com](http://www.BARDEP.com)



Before use, consult product labels and inserts for any indications, contraindications, hazards, warnings, cautions and use.

Bard and the stylized heart are registered trademarks of C.R. Bard, Inc. or an affiliate. Orbiter and Easy-Mate are trademarks of C.R. Bard, Inc. or an affiliate. All trademarks or registered trademarks are the property of their respective owners.  
© 2010 C.R. Bard, Inc. All rights reserved. LT04Z0004/Rev. 01/12-2010